

# “GO LIST”

## APPROVED MATERIALS LIST FOR MONOPLACE CHAMBER



**Concern categories: Energy Source, Fire/Flashpoint, Pressure, Pharmacologic, Patient Comfort, Acrylic Damage, Infection Control, and Choking Hazard**

MATERIAL	DESCRIPTION	CONCERN	REFERENCE
BP (NIBP) cuff with tubing	<b>Use of pass-through to connect to Oscillomate 1630 monitor outside chamber.</b>	Energy	NFPA 99 14.2.7.6
Breathing system equipment (built in)	Supply hose with door connection, demand valve mask, mask, face tent, trach collar, non-rebreather masks	Comfort, Pharmacologic	NBS OM
Catheter, (indwelling urinary cath or wound drainage system)	Flexible tubing with collapsible reservoir <b>Empty excess drainage, purge air</b>	Comfort, Pressure	NBS P&P
Cloth restraints	No velcro or metal components <b>If velcro present TAPE OVER EACH FACE OF VELCRO, if metal present TAPE OVER EXPOSED SURFACES</b>	Energy, Fire, Acrylic Damage	NBS OM P&P
Contact lenses (soft and gas permeable)		Comfort	SN 0807
Cotton gauze dressings	Pads, packing, wraps, island-type with non-stick surface	Comfort, Pharmacologic	NBS P&P
Diaper (adult/infant) /Kotex	<b>Should remain against patient skin, moist, no velcro closures</b>	Comfort, Infection	NBS P&P
Dressings (aqueous) or topical medication	Hydrocel	Pharmacologic, Fire, Comfort	NBS P&P
ECG and pressure line monitoring cables	Biomed approved/maintained <b>Use of pass-through to connect to monitor outside chamber.</b>	Energy	NFPA 99 14.2.7.6
Eyeglasses	Plastic or metal composition, lenses	Comfort	NBS SN 0903
Feeding tube (Dobhoff)	Flushed and clamped	Comfort, Pressure, Pharmacologic	NBS P&P
Gowns, hospital issue	100% cotton, unit supplied, pocket less, no metal snaps or Velcro, uncontaminated. <b>Must check for concealed items</b>	Comfort, Fire	NFPA 99 14.3.1.5.4 NBS SN 0503
Heimlich Valve	Attached to indwelling chest tube and sterile glove or urinary drainage bag	Pharmacologic, Comfort, Pressure	NBS OM
IV extension tubing for Monoplace Hyperbaric chamber	Hospira <b>Use of pass-through to connect to hyperbaric IV pump outside chamber</b>	Pressure	NBS OM P&P
Mattress, wedges, pillows	Only those recommended by chamber manufacturer	Comfort, Fire, Pressure	NBS P&P
Nasogastric tube	Un-clamped with end loosely covered	Comfort, Pressure	NBS P&P
Nebulizer equipment (in-chamber)	Disposable nebulizer, 5-in-1 adapter, liquid medication	Pharmacologic, Comfort	NBS P&P
Plastic drink cup	No-spill style, vented, unit supplied, only approved unit supplied fluid	Comfort, Pressure, Fire, Pharmacologic	NBS SN
Plastic tape	No paper or silk tape	Fire, Energy	NFPA 99 14.3.1.5.4.1
Pressure infuser with spacer	Ethox Surgi-Press, oxygen-safe lubricant (silicon base)	Pressure, Pharmacologic	NFPA 99 14.3.6.2.1.4
Sheets, pillow cases, towels, wash cloths, blankets	100% cotton, unit supplied	Comfort, Fire	NFPA 99 14.3.1.5.4.2
TCOM electrode with cord	Radiometer – Max 28v and 4w in class B chamber <b>Use of pass-through to connect to monitor outside chamber</b>	Energy	NFPA 99 14.2.7.6.1, 14.2.7.6.1.1
Urinal, bedpan	Disposable, plastic	Comfort	
Ventilator breathing circuit (hyperbaric)	Sechrist Model 500A <b>Use of gas port to connect to control module outside chamber</b>	Pressure, Pharmacologic	NBS OM P&P

**Reference Terminology:**

NFPA; National Fire Protection Association 99 2012 Edition  
 NBS SN; National Baromedical Services Safety Notice  
 NBS OM; National Baromedical Services Operations Manual  
 NBS P&P; National Baromedical Services Policy and Procedure Manual