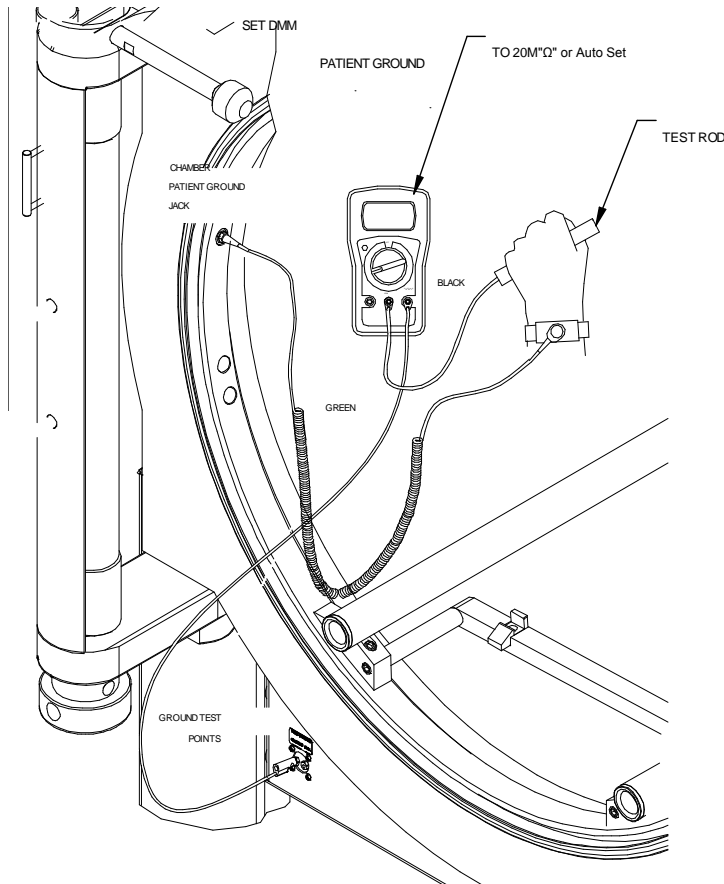


**PATIENT GROUNDING** (Refer to Figure 10):

Patient ground jack located on Door Ring

**Figure 10**

- Step 1: Wear wrist strap.
- Step 2: Connect wrist strap plug to Chamber Patient Ground jack.
- Step 3: Connect the green test cable to V•Ω•mA jack of the Sechrist provided DMM.
- Step 4: Connect the other end of green test cable to the Chamber Ground Test point jack labeled Chamber.
- Step 5: Connect the black test cable to COM of the DMM and the other end of the test cable to the Patient Ground Test rod.
- Step 6: Set the DMM dial to 20MΩ. or Auto setting if available.
- Step 7: Grasp the Patient Ground Test rod with the hand wearing the wrist strap.
- Step 8: Obtain resistance reading from DMM display.

Note: Digital MultiMeter values greater than 1 meg ohm is a passing value.